

PRODUCT INFORMATION

EV4193 / EV4346

Batch 588186 / 588187 Onwards

MultiSTAT Drugs of Abuse Urine / MultiSTAT Drugs of Abuse Urine (CE)

The Benzodiazepines I assay has been updated to a monoclonal antibody. The Cross-Reactivity profile for this assay has been updated on the Instructions for use. All other performance claims are equivalent to the current assay.

Please see below for details of the new Benzodiazepines I assay Cross-Reactivity profile information.

The changes are effective from batch 588186 / 588187 onwards and do not apply to material previously released.

Benzodiazepines I Assay		
Compound	Approximate % Cross Reactivity Concentration to Read Positive (ng/ml)	Approximate % Cross Reactivity Concentration to Read Positive (ng/ml)
Oxazepam	150	100
Alprazolam	15	1000
Estazolam	39	384.6
Nordiazepam	78	192.3
Diazepam	120	125
Oxazepam Glucuronide	120	125
Midazolam	120	125
Clobazam	225	66.7
Bromazepam	270	55.6
Temazepam	300	50
Temazepam Glucuronide	300	50
Chlordiazepoxide	300	50
Alpha-Hydroxyalprazolam	405	37
N-Desmethylflunitrazepam	600	25
Nitrazepam	600	25
Phenazepam	898	16.7
Triazolam	898	16.7
Lorazepam	1049	14.3
Lorazepam Glucuronide	1200	12.5
Flunitrazepam	2113	7.1
Medazepam	2380	6.3
Lormetazepam	4545	3.3
Clonazepam	6000	2.5
Prazepam	25,000	0.6
2-OH-Ethylflurazepam	20,000	0.75
Flurazepam	20,000	ND
Etizolam	20,000	ND
7-Aminonitrazepam	20,000	ND
7-Aminoclonazepam	20,000	ND
7-Amino Flunitrazepam	20,000	ND